

Lucent LED Operatory Light Quick Reference Installation Guide

ASSEMBLING THE HEAD TO THE ARM

Remove adaptor Bushing

Attach bushing to light head





ELECTRICAL CONNECTIONS


Connect wire harness

Secure light head to arm





Light Handles

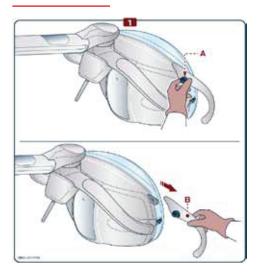

Switch or	/ off / Ad	justment
-----------	------------	----------

- To switch the lamp On and Off, press and release the switch to the LEFT or RIGHT side. When the lamp is switched On / Off, the device will beep (once). DON'T push and pull the switch. It only moves in a left and right direction.

Signature

Date

Right & Left Light handle
Light switch toggle extension
Light shield wrench
Light shield wrench

Light bushing pre installed into arm.